

Information is Power: Using Real-Time Data to Track Blood Glucose Trends of Cellnovo Mobile Diabetes Management System Users

Yasmin Friedmann PhD¹, Olivia Hautier-Suply MD², Julian Shapley PhD BSc³

1. Data Scientist, Cellnovo, UK 2. Medical Affairs Manager, Cellnovo, UK 3. Chief Scientific Officer, Cellnovo, UK

Background and aim

The Cellnovo Diabetes Management System comprises an insulin micro pump, an activity monitor, a mobile-enabled wireless touch-screen handset with integrated blood glucose meter, and automatic connectivity to a secured server.

A retrospective analysis of data stored on the Diabetes Management portal and collected between January 2013 and September 2017 was undertaken. Here we highlight some of the results.¹

Methods

We retrieved a cohort of 166 patients who had used the pump for at least 6 months and had recorded their BG at least 3 times a day on average. The median follow-up time for this cohort is one year (range 6-41 months). Where we compared first and last months' data we used paired Student's t-test or Wilcoxon rank-sum test as appropriate. $p < 0.05$ was taken to indicate significance. We present a comparison of first and last month data of:

- HbA1c values where available
- Mean and standard deviation (SD) of BG
- Number of hypoglycaemic events per week

Results

- Among 30 adults with HbA1c values available (mean follow-up time 10 months) we found a significant reduction in HbA1c of 0.54% (from 7.67% to 7.13%, $p=0.002$). The higher the value for HbA1c at baseline, the greater the improvement ($p=0.0007$) (Figure 1)
- Glycaemic data available for the 166 patients (median follow-up time of one year), demonstrated:
 - Of 143 adults, 23 (16.1%) had a significant improvement in mean BG levels of between 13.4mg/dl and 158.1mg/dl
 - BG variability decreased between 0.4mg/dl and 51.6mg/dl in 53 (37%) adults (Table 1)
 - In adults ($n=143$), there was a significant decrease in the number of hypoglycaemic events; 1 event less per week (from 3.4 to 2.4 events/week, $p<0.00001$) with 93 (65.03%) patients having fewer events (Figure 2)
 - In children ($n=16$) there is some decrease in the number of hypoglycaemic events per week between the first and last month of data collection (from 3.7 to 3 events/week, $p=0.27$) with 11 (68.75%) children having fewer hypoglycaemia events
 - In the adolescent group, hypoglycaemic events decreased almost significantly (from 3.8 to 2.3 events/week, $p=0.05$)

Figure 1: Overall HbA1c outcome
Mean follow up duration: 10 months, $n = 30$ adults


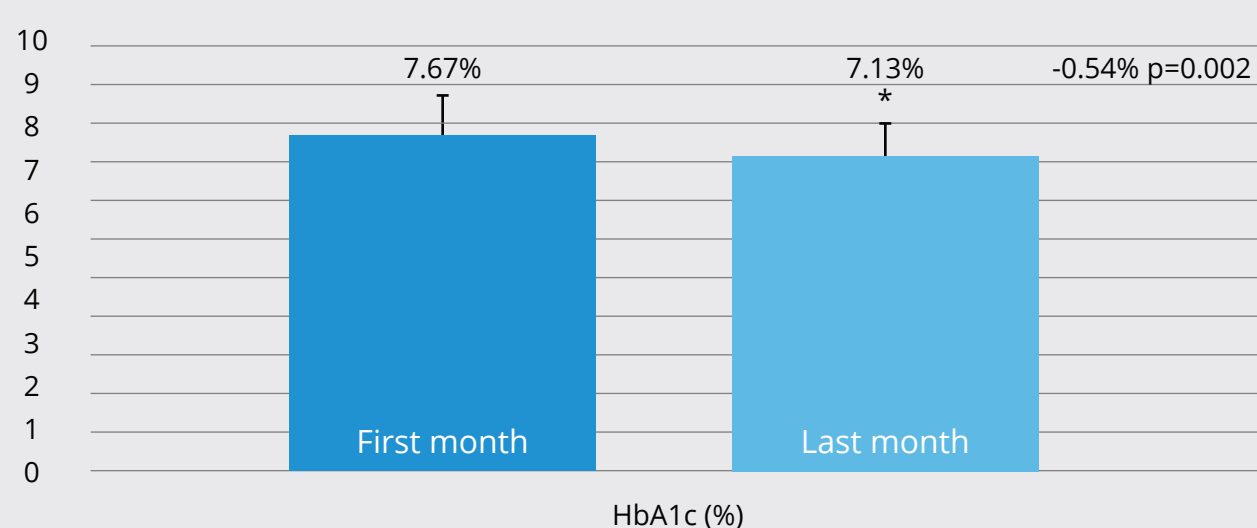


Figure 2: Hypoglycaemia frequency, events/week

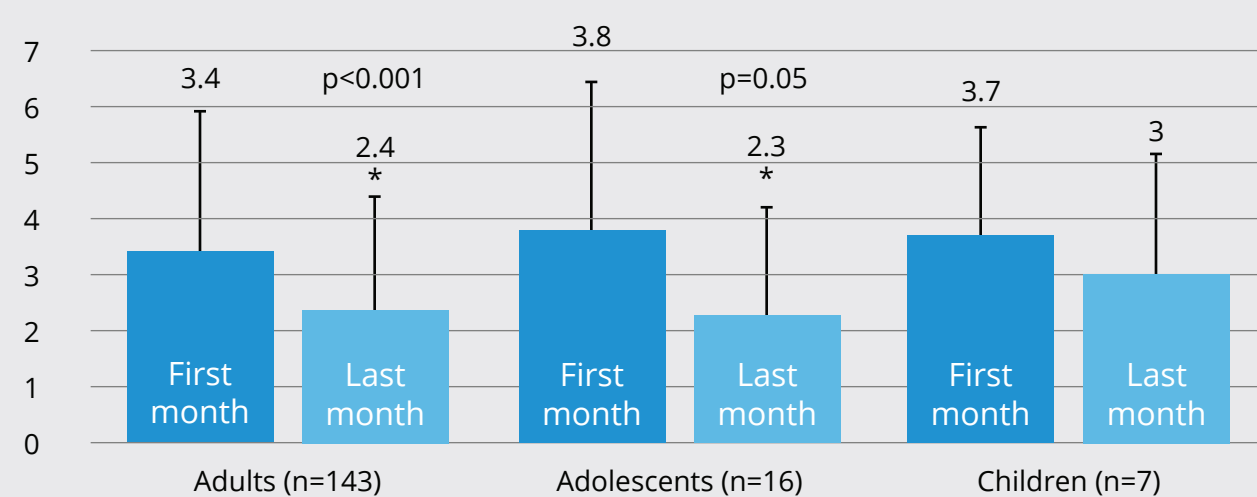

Conclusion

The preliminary results from the data of a cohort of Cellnovo System users showed real-life data that indicates a decrease in HbA1c level and in the number of hypoglycaemic events in HbA1c for adults. Similar results were demonstrated in a previous case study².



Table 1: Clinical Parameters from Baseline to Final Month of Data Analysis (Mean±SD)

	Children (n=16)			Adolescents (n=7)			Adults (n=143*)		
	Baseline to and of first month	Last month	p	Baseline to and of first month	Last month	p	Baseline to and of first month	Last month	p
BG (mg/dl)	178±23.6	182.6±32.7	0.55	159.7±32.3	166±36.3	0.46	162.7±30.3	169±30	0.01
Standard deviation of BG (mg/dl)	83±19	86±13.6	0.4	75.6±8	75±24	0.9	72.6±16	76±17	0.007

SD = Standard Deviation
BG = Blood Glucose
* some data values missing from System

References

1. Hautier-Suply O, Friedmann Y, Shapley J. A retrospective review of the clinical characteristics and blood glucose data from Cellnovo System users using data collected from the Cellnovo Online Platform. *European Endocrinol*, 2018;14(1):Epub ahead of print
2. Kelly P. Achieving effective glycaemic control using an insulin micro-pump. *British Journal of Community Nursing*. 2017. 22; 2:66-75